

Twinky Star

THE COLOURED RESTORATIVE
WITH GLITTER EFFECT FOR DECIDUOUS TEETH

Twinky Star

THE COLOURED RESTORATIVE WITH GLITTER EFFECT FOR DECIDUOUS TEETH

The top priority of healthy deciduous teeth is to ensure the chewing function of the growing child. The teeth in the first dentition are also extremely important for language development, as a placeholder for the teeth in the second dentition, for the development of the chewing function and normal-, progressive skeletal growth in the oral maxillofacial area.

In addition to reducing caries activity through advice to the parents and appropriate prophylactic measures, a tooth already affected by caries must be adequately treated by a dentist. In doing so, the little patient is often a real challenge for you and your staff. In addition to patience, a restorative is required that supports your therapy efforts and can additionally encourage positive compliance from the child and parents alike.

The little patient – a challenge

With the treatment of children, you are often confronted with fear, nervousness, impatience and mistrust. You have nevertheless found methods of circumventing these stress factors that can make your work extremely difficult. A component of successfully treating a child is also the restorative you choose:

Twinky Star - the coloured compomer with glitter effect

Twinky Star, the coloured restorative with glitter effect, was developed especially for children. You can include your little patients in the treatment procedure by giving them the choice of different colours to choose from. Children have the feeling that they can actively influence the treatment when they select the colour for themselves. Fear and impatience thus can be pushed a bit into the background. The treatment can therefore


proceed with less stress, which has a positive effect on the compliance of the child. Moreover, the little patients develop a lasting interest in the care of their teeth so that the new, coloured fillings remain "pretty" for as long as possible. (Dr. Schäfer, Christine; ZMK Zahnheilkunde, Management und Kultur; Aug.12/2004; S. 866-868). And the health of the entire deciduous dentition is benefited in the long run.

Proven VOCO technology

In an independent survey, American dentists were questioned about the clinical use of Twinky Star. The characteristic mentioned most often about Twinky Star is the widespread acceptance from children and the successful colour selection with actual colour samples. Twinky Star's good consistency and simple processing were also praised.

Boys and girls alike will be enthusiastic about choosing their favourite colour.

Twinky Star offers the following colour palette:

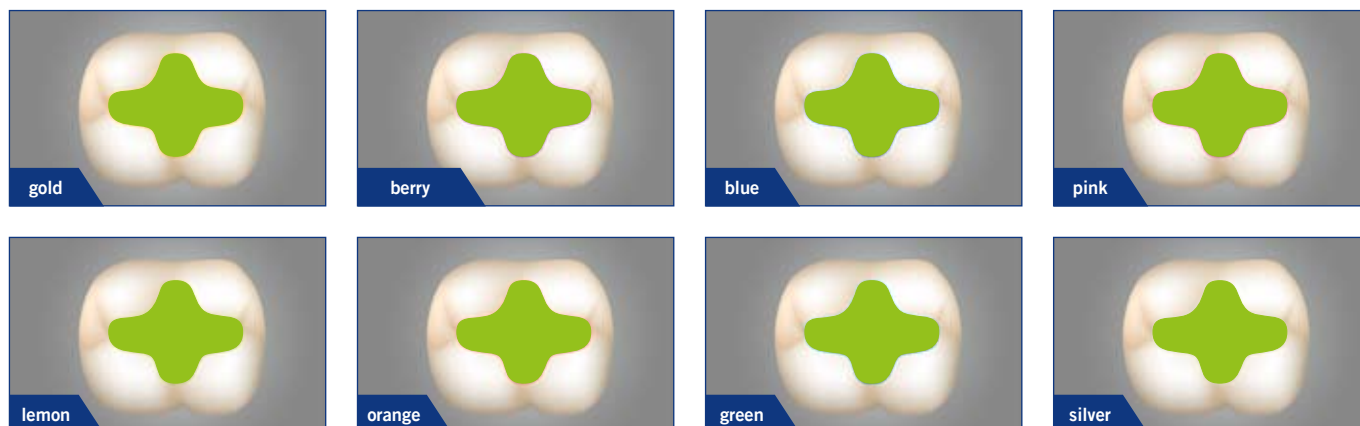

TECHNICAL DATA					
Transverse strength	116	MPa	Polishability	Excellent	
Filler content	77,8	Gew.%	Radiopacity	220%	AI
Abrasion	15,4	µm	Shade stabilityt	No decolourization	
Compressive strength	316	MPa	Minimal polymerisation shrinkage	2,0	Vol.%
Adhesion to enamel/dentine	27,3/24,1	MPa			

Twinky Star is based on the proven VOCO compomer technology. Compomers are suited best for paediatric restorative therapy. In addition to good physical properties, such as low abrasion, they are also characterised by their high biocompatibility, good adhesion and marginal adaption as well as a supplemental fluoride release. The restorative is not only coloured, it is also safe and durable .


in only 4 steps in a total of ca. 35 seconds. Conditioning the tooth is unnecessary. Good treatment results are achieved with Futurabond M even under difficult conditions.

Twinky Star is available in convenient Caps. This system allows the restorative to be quickly and directly applied into the cavity.

The quick treatment – Twinky Star and Futurabond M

Since children as patients only tolerate treatment for a limited amount of time, the dental procedure must take place in a short window of time. A speedy restoration is possible when combining Twinky Star with Futurabond M, the self-etch bond from VOCO, and Twinky Star. The light-curing, one-component, self-etch bond is simple and quick to use. This takes place

Compomer restoration – also an alternative for paediatric dentistry

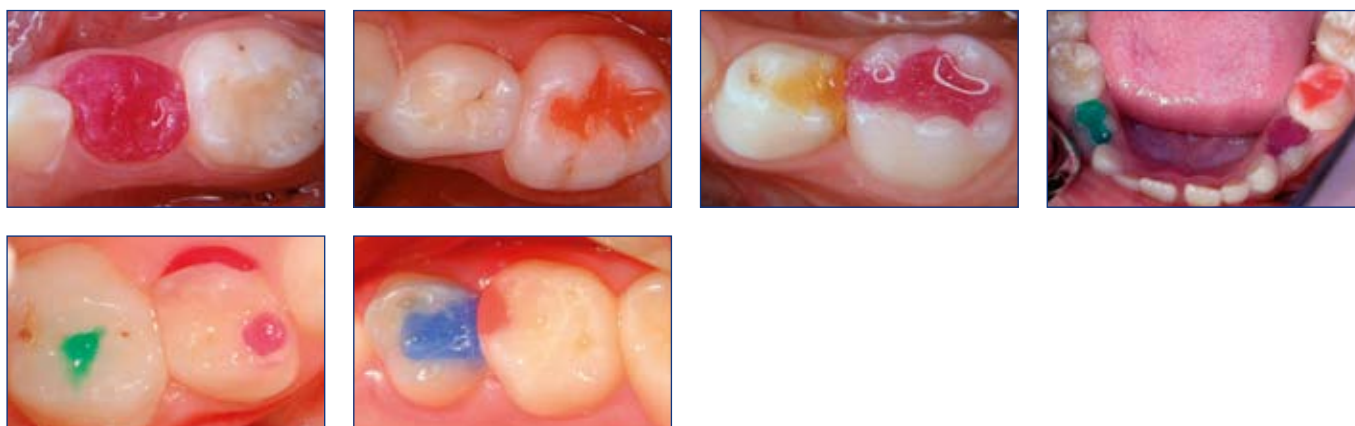

Twinky Star

THE COLOURED RESTORATIVE FOR DECIDUOUS TEETH

Indications

Restoration of deciduous teeth

Presentation

REF 1680	Set 40 x 0.25 g Caps in different colours, colour guide
REF 1679	Set (40 x 0,25 g Caps) + Futurabond M 50 <i>Single Dose</i>
REF 1681	Gold
REF 1682	Silver
REF 1683	Blue
REF 1684	Pink
REF 1685	Green
REF 1686	Orange
REF 1687	Lemon
REF 1688	Berry
REF 1689	Twinky Star colour guide

Advantages

- Eight attractive, glitter effect colours
- Quick and simple application from caps
- Very high acceptance by children
- Children lose their fear of visiting the dentist
- Parents can more easily get their children to go to the dentist
- Excellent polishability
- Fluoride release against secondary caries
- Colour guide in original colours
- Material is based on proven VOCO compomer technology
- Excellent material properties
- Restorative keeps intact until exfoliation
- Eight years of clinical success



VOCO GmbH
 P.O.Box 767
 27457 Cuxhaven
 Germany

Tel.: +49 (0) 4721-719-0
 Fax: +49 (0) 4721-719-140

info@voco.com
 www.voco.com

Available from: